

Reset your Coronet to Factory Settings

1. Connect the coronet to the App

Open the Well Red app.


Select *Coronet*. The app will then show a page similar to this:

Select the **i** just under the green bar – the information icon.



The app will show a page similar to this, with lots of details showing your Coronet's settings.


Select *Reset to factory setting*

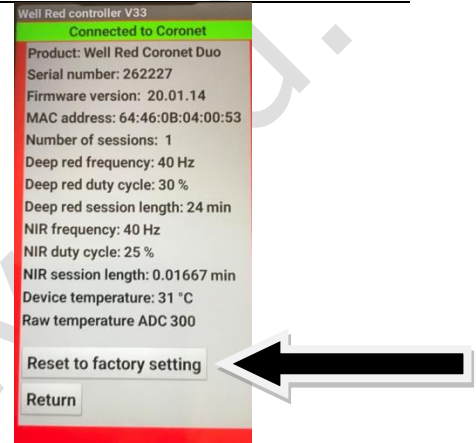

You will be asked if you have a Mono or a Duo coronet.

Select your Coronet model.

Mono – 1 wavelength only
Duo – 2 wavelengths.

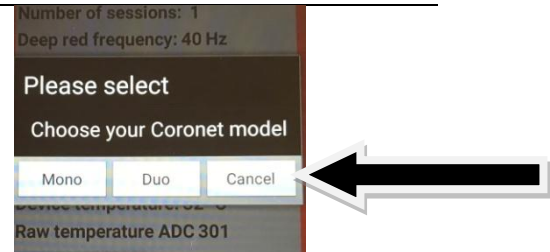

A few seconds after you have selected your Coronet model, the Coronet will blink once.

This means that the new setting is in place.


The words *Coronet reset to default settings* appear briefly



Reset your Coronet to Factory Settings

Check the information page again, and the settings should have changed.

This photo shows the Duo setting has been restored.

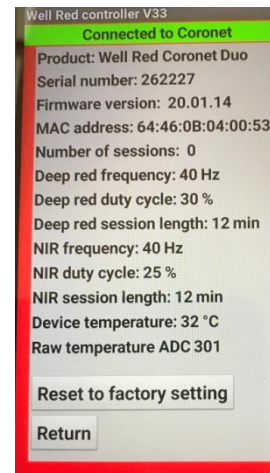

Well Red Pty Ltd.